

Changing Hooks

dreamTAP comes with several hooks. They can be changed in the practice. Here is how..

Written By: Patrick Tessier



INTRODUCTION

dreamTAP comes with three hooks. You can change them in the practice. Here is how to do it.



TOOLS:


- [1.5 mm Locking Screw Hex Key](#) (1)




PARTS:

- [dreamTAP hooks](#) (1)



Step 1 — Removing the Locking Screws

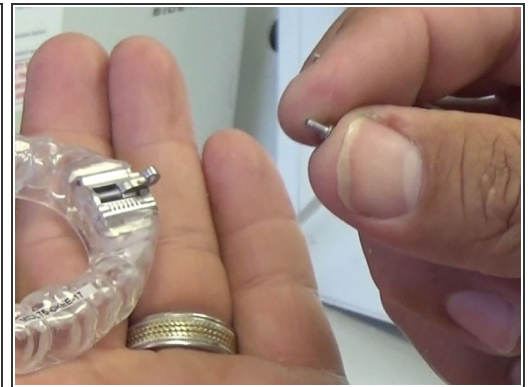


- The appliance comes with two hex key drivers. The locking hex key is a simple 1.5 mm key. The Adjustment key has the black handle
- Use the 1.5 mm hex key to remove the two locking screws.


 Don't drop and lose the screws. They are quite small .

Step 2 — Remove the front plate

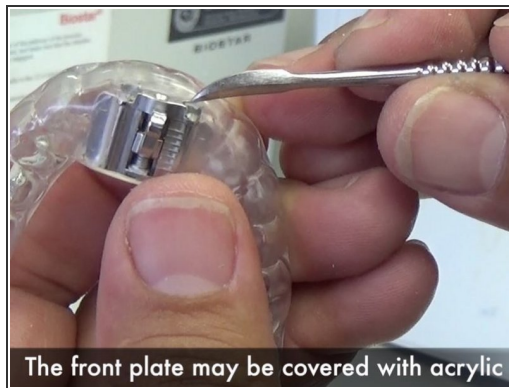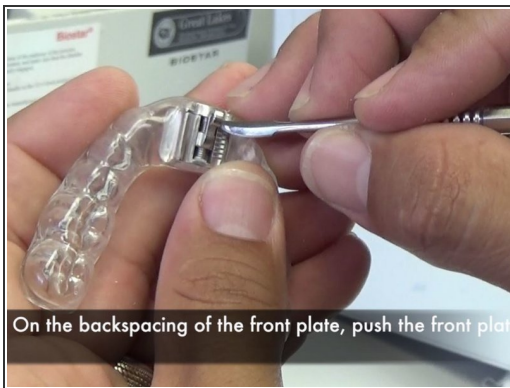


- Push the front plate off by using a small hand tool to pop it off
-  There may be some acrylic covering the plate. You will need to remove with a burr or blade.
- Once the front plate is off, remove the hook and screw assembly.

Step 3 — Complete the disassembly

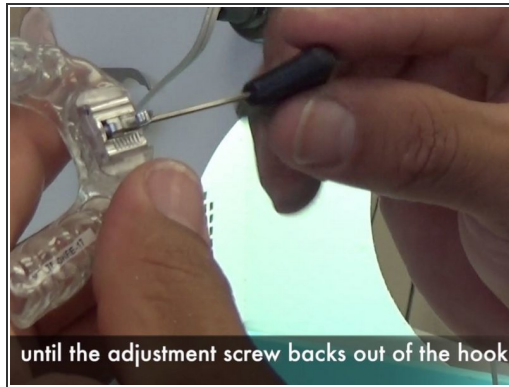


- There is a small clear/white washer on the end of the screw. Remove and store. We will need it when putting back together.
- Next, put the hood and screw back into the assembly. It will hold the hook. Now turn the screw until it fully backs out of the hook.
- Now you can pull the screw and hook out. It's fully disassembled.

Step 4 — Reassemble with the new hook

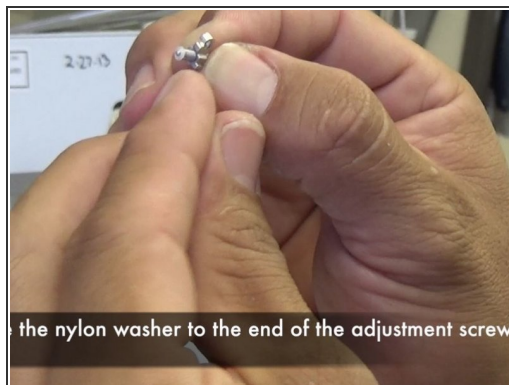

- Insert the new hook into the assembly and then turn the screw into the hook.
- ⓘ Remove the hook/screw and place the white/clear washer on the end of the screw. It's pretty small so be careful.
- Put the new hook/screw/washer into the base assembly.

Step 5 — Replace the front plate and test new hook

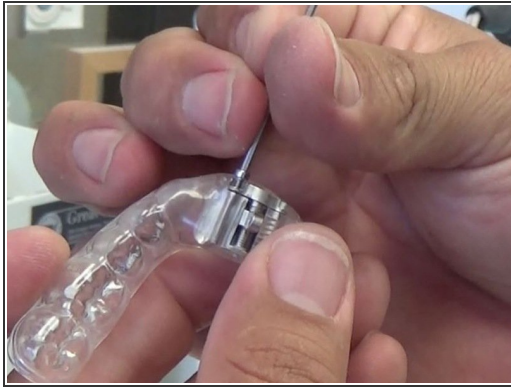

- Place the flat side of the plate against the base and replace the two locking screws.
- ⚠ Be sure not to over tighten. This can strip the threads.
- ℹ Make sure there is no gap between the front plate and the base.
- Finally, test the hook movement to insure it moves freely.