

Directions for Daily Use

Written By: Patrick Tessier



Step 1 — Daily Use



- **BEFORE EACH USE:** • Make sure the air path through the mask is clear. • Inspect the vent holes. Do not use if blocked. • Inspect the mask for damage. DO not use if there is any visible deterioration (tears, cracks, etc.), and seek replacement part(s).
- Connect the CPAP hose to the swivel connector on the top of the inlet tube. Then turn on the machine.
 - ⓘ It is easiest to assemble and disassemble the Swivel Connector at an angle.
- Place your hands around your nose and feel for any air leaks. If there is a leak from the pillows, loosen the nut and adjust as necessary. Note that exhaled air will escape from the holes on the front of the mask.

Step 2 — Homecare Instructions



- **After Each Use:** Rinse the appliance with cold water and let it dry out of direct sunlight.
- **Weekly:** Hand wash the Chin Strap and/or Headgear if used, following the same procedure as above. Remove the Pillows from the Mask Frame Assembly and remove the Headgear. Place the Pillows and Mask Frame in lukewarm water that contains a mild soap. Scrub Mouthpiece with a soft toothbrush.
- ⓘ NOTE: Do NOT soak for more than 10 minutes.
- Remove the mask components and rinse them thoroughly in fresh water, ensuring that all soap residue is removed. Leave all parts to dry out of direct sunlight.
- ⚠ Do NOT wash the mask in the dishwasher.
- ⚠ Do NOT clean the mask with products that contain chlorine, bleach, moisturizer, antiseptic, anti-bacterial agents or alcohol.
- ⚠ Do NOT wash your mask in hot water as this may change the fit of your Mouthpiece.
- ⓘ NOTE: Failure to follow the cleaning instructions may compromise the performance and safety of the mask.