

15 mm Protrusion Range

Only the dreamTAP allows for up to 15mm of protrusion range.

Written By: Patrick Tessier

Protrusion Range

- 15 mm w/ three hooks

Long Hook



Medium Hook



Short Hook




Starting Position



Max Retruded

Max Protruded

Step 1 — The screw mechanism

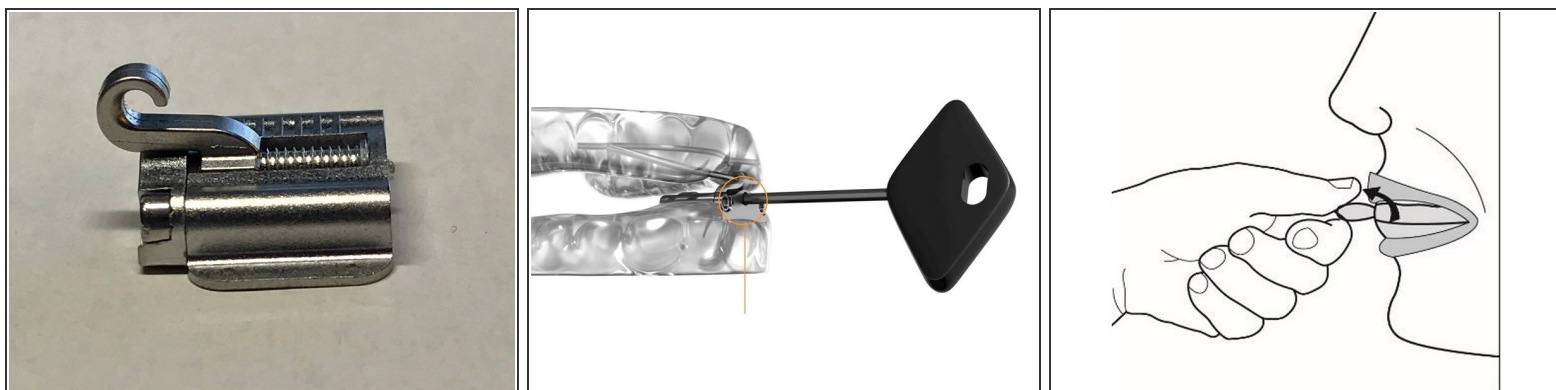

- The dreamTAP advancement mechanism utilizes a custom screw device.
- Using the supplied key, you can advance the hook 5mm.
- A half turn, counter clockwise facing the patient, will advance 0.25 mm

Step 2 — Three Hooks for 15mm range



- Each case comes with all three hooks. Each hook allows for an additional 5mm range.
- Typically, the medium hook is used in the appliance and the long and short hooks are supplied for later use.
- A clinician can easily change hooks in the practice, in about 5 mins or less.