


New Restorations under existing TAP appliance

Written By: Patrick Tessier

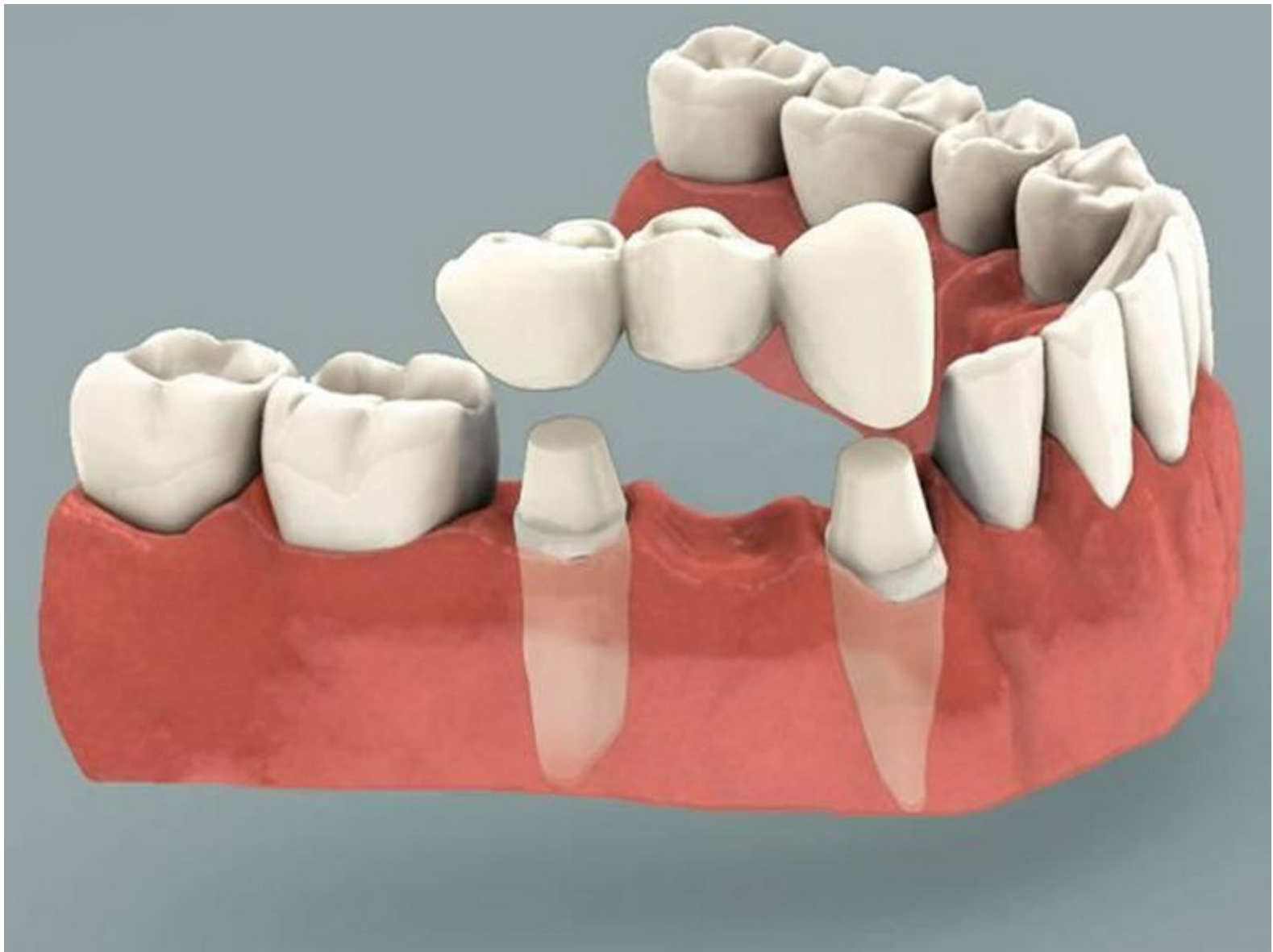

INTRODUCTION


It's fairly easy to adjust a TAP appliance for new restorations. The key ingredient = ThermAcryl



PARTS:

- [ThermAcryl Material](#) (1)
-

Step 1 — Adjusting a ThermAcryl lined appliance

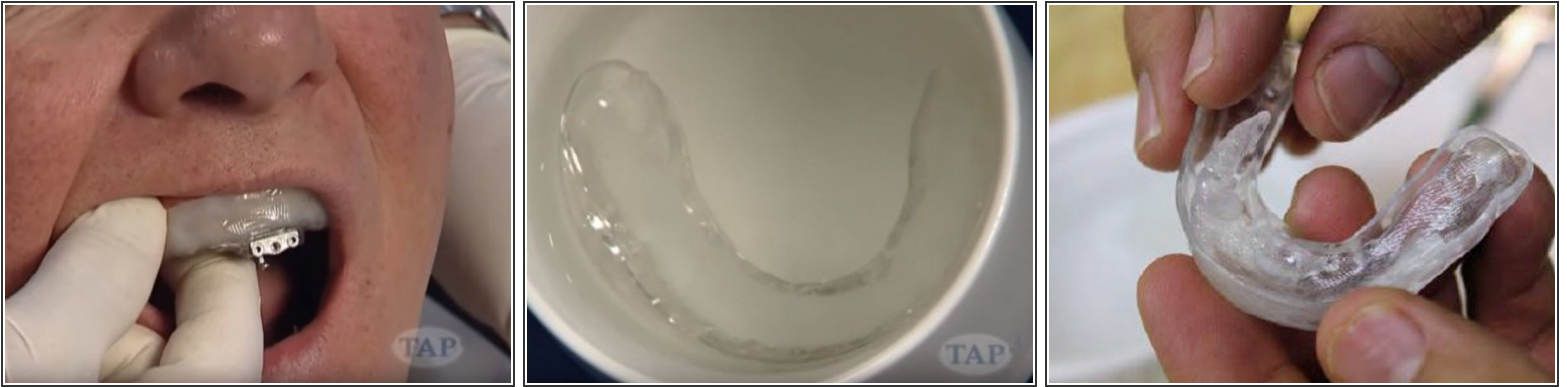

- If you ordered the appliance with ThermAcryl liner, you're in luck.
- If the adjustment is minor, for instance a single crown, you just need to heat up the appliance and let the ThermAcryl do its magic. The adjustment procedure is the same as the fit procedure. [Delivering a dream TAP ThermAcryl guide.](#)
- If the adjustments are major, for instance with a three unit bridge, to replace a missing tooth, you can reline the appliance and basically start from the beginning. Here is the [Relining with ThermAcryl Guide.](#)

Step 2 — Adjusting a Triple Laminate tray



- If you ordered the appliance as a TL, with the Urethane liner, you can still adjust for new restorations.
- If the new restoration is a single crown, you may be able to just reuse the appliance. But the crown would have to be an exact match.
 - If the new crown is slightly larger, you can remove some urethane liner to make room. Be careful and don't remove too much. It cannot be replaced and you may lose retention.
- The more difficult option would be to completely strip out the urethane liner and replace with ThermAcryl. [Here is the guide for relining with ThermAcryl.](#)