

Short teeth and retention issues

Written By: Patrick Tessier



INTRODUCTION

This is a case for ThermAcryl lining



PARTS:

- [ThermAcryl Material](#) (1)
-

Step 1 — Order the appliance with a ThermAcryl liner



- When presented with a case with short teeth, think **ThermAcryl**. Retention will be difficult at best for any other type of tray system.
- When seating a case with a ThermAcryl liner, you will be making the final fit in the chair, during delivery.
- The material will flow against 100% of available tooth surfaces. And with a high coefficient of friction it "**Sticks**" really well.
- Here is the guide on [delivering a dreamTAP ThermAcryl](#) that illustrates the procedure.